
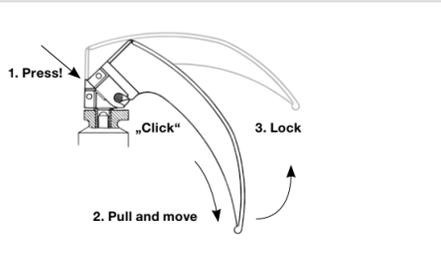
HEINE Classic+ F.O. Laryngoscope Blades

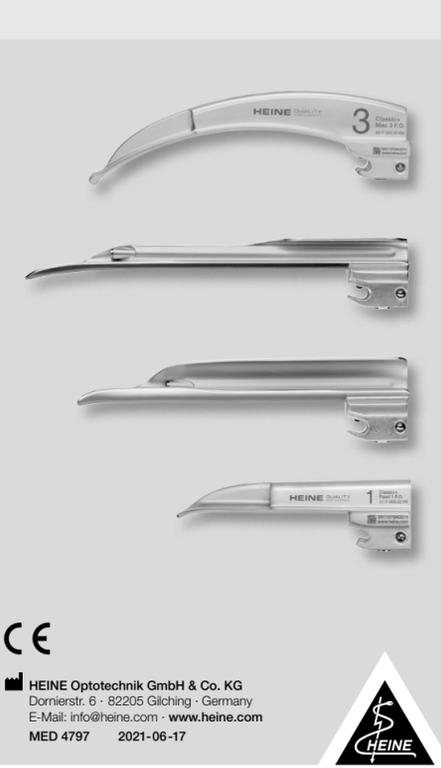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	Please read and follow these instructions for use and keep them for future reference.
---	---

Intended Use

The HEINE laryngoscope blade is intended for the transient use with laryngoscope handles according to ISO 7376 for laryngoscopes for tracheal intubation.

It should only be operated by qualified medical personnel within a professional healthcare facility or emergency medical services environment.

⚠️ **For U.S. only:**
Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

- ⚠️ CAUTION!** This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations with moderate or severe extent. (Background color yellow; foreground color black).
- 🔦 NOTE!** This symbol indicates valuable advice. Notes are important, but not related to hazardous situations.

Setting up

Fix the blade into the working position (see fig.).

Operation

Before use, check for sufficient light emission after coupling with a laryngoscope handle.

The setting up and the operation of the HEINE laryngoscope handles are described in separate instructions for Use.

⚠️ Only use a laryngoscope handle/blade combination with ISO 7376 standard to ensure proper functioning.

The HEINE laryngoscope blades correspond to the ISO 7376 standard. If non HEINE laryngoscope handles are used, component compatibility and functionality are to be checked before each use!

Only trained personnel are allowed to use a laryngoscope for intubation purposes.

HEINE laryngoscopes are designed for use < 5 minutes.

Check the correct operation of the device before each use! Do not use the device if there are visible signs of damage (e. g. rough surfaces and edges etc.)

Before each use, check the condition of the connector for damages, wear and tear. If defects exist, do not use the laryngoscope blade and replace it with one that is in working order.

We strongly recommend a functional backup of laryngoscopes in an emergency situation.

Hygienic reprocessing

The instruction is available:

- at www.heine.com
- in a paper version which you can request from the address listed

Maintenance

The device ist maintenance free.

Service

The device has no components serviceable by the end-user.

General Notes

🔦 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 12 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

General Warnings

⚠️ This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

Disposal

Perform a hygienic reprocessing before disposal.

Please observe the relevant state-specific disposal regulations.

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